# Moji AI Cilt Test Cihazı - Max

## Kullanma Klavuzu

# İÇİNDEKİLER

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
# Superficial Wrinkle Test Image

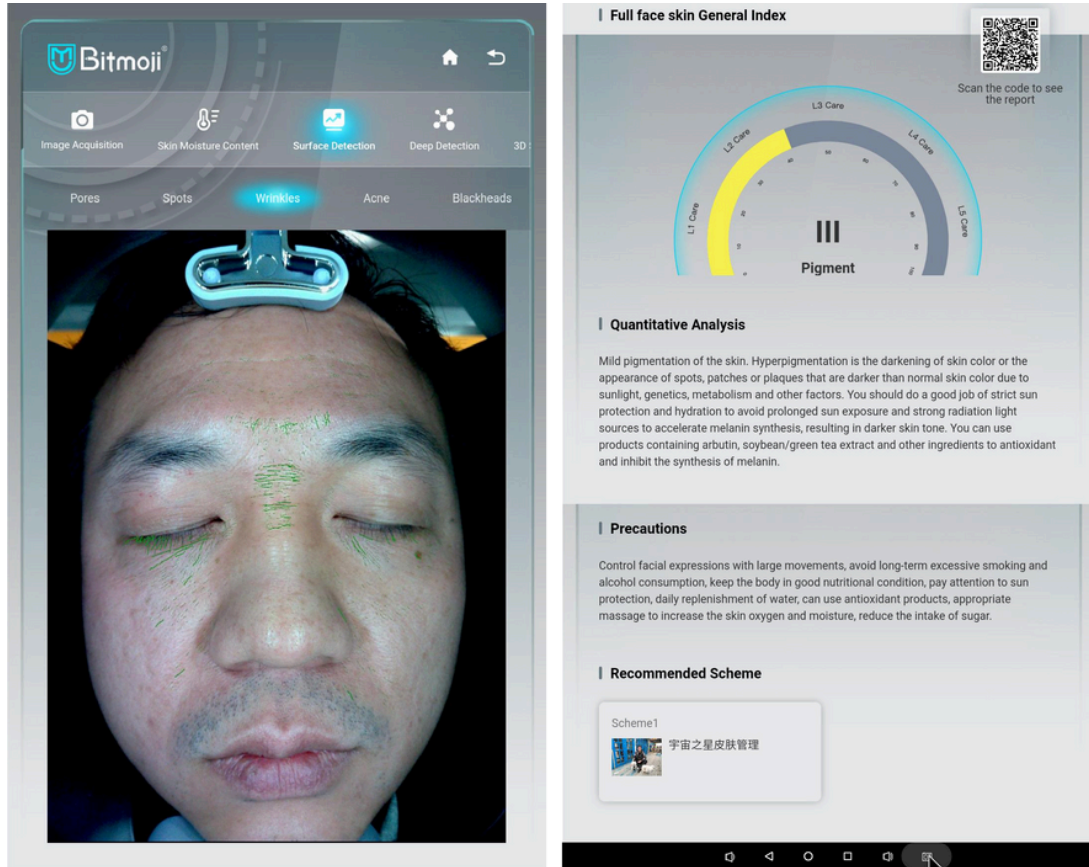**Test basis:** In the polarized light mode, the development clarity of the skin surface texture can be improved, the skin smoothness, fine lines and wrinkles can be clearly observed, and corresponding nursing-level analysis, precautions and project product recommendations can be made.

**Look at the picture:** The icon of green short thin line in the picture indicates that there are dry lines and fine lines or the loss of collagen in this part.

**Level:** The Level 3 in yellow in the picture indicates the problem of fine lines and dry lines is mild to moderate.

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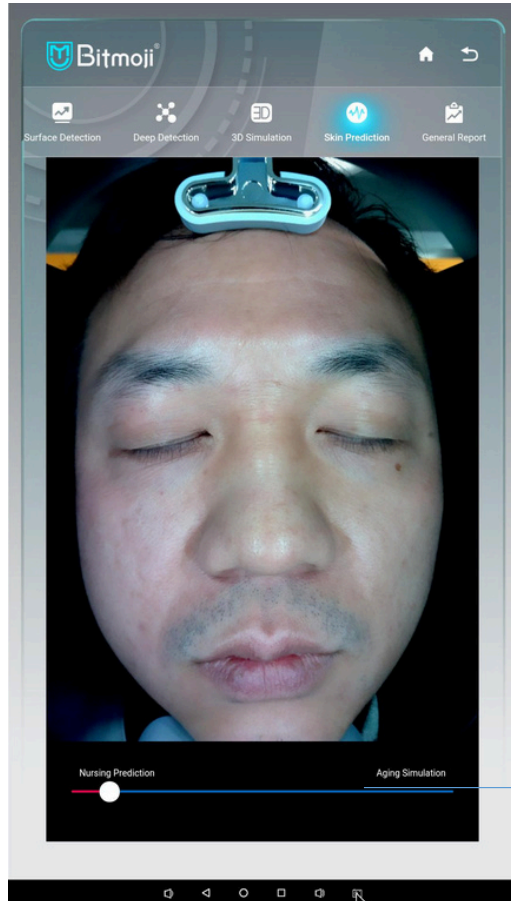
# Skin Aging Prediction

**Test:** Visual demonstration of images to predict the aging or beautification state of the skin after 5 to 10 years.

**Operation:** You can drag the image left and right. Dragging from left to right of the image demonstrates the states from youth to aging.

**Look at the picture:** You can make a crisis judgment by dragging the skin aging image to the right through the picture icon. If you don't take care of your skin properly, it will turn into an aging look.

You can move the image left and right. Moving from left to right of the image demonstrates the states from young and aging.

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Add Goods Batch Delete

product keywords Search

ID	Product Image	Product_name	Price	Efficacy	Operation
159		超气泡活肤仪	6000.00	clean,Replenishment,Rejuvenation	<a href="#">Edit</a> <a href="#">Delete</a>
158		水精灵美容仪	6800.00	Replenishment,Pore shrinkage,Rejuvenation	<a href="#">Edit</a> <a href="#">Delete</a>
157		时光机美容仪	16800.00	compact,repair,antifading	<a href="#">Edit</a> <a href="#">Delete</a>
156		宇宙之星皮肤管理	20000.00	antifading,Anti wrinkle,compact	<a href="#">Edit</a> <a href="#">Delete</a>

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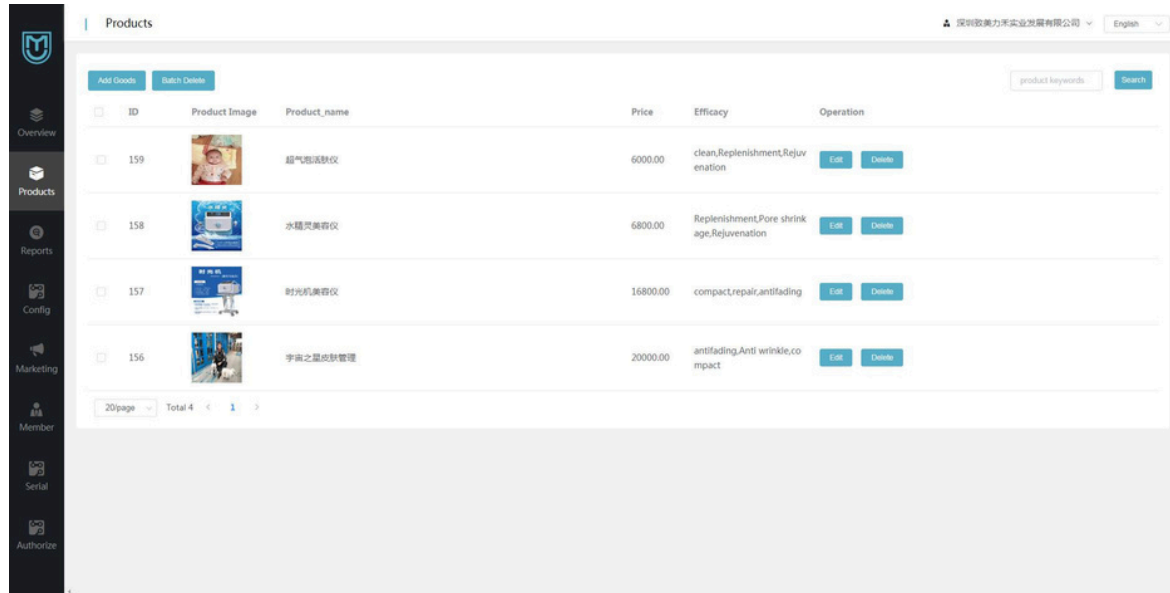
ID	Product Image	Product_name	Price	Efficacy	Operation
159		超气超活肤仪	6000.00	clean,Replenishment,Rejuvenation	<a href="#">Edit</a> <a href="#">Delete</a>
158		水精亮肤仪	6800.00	Replenishment,Pore shrinkage,Rejuvenation	<a href="#">Edit</a> <a href="#">Delete</a>
157		时光肌美容仪	16800.00	compact,repair,antifading	<a href="#">Edit</a> <a href="#">Delete</a>
156		宇宙之星点阵管理	20000.00	antifading,Anti wrinkle,compact	<a href="#">Edit</a> <a href="#">Delete</a>

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Product\_price

\*Product Image

\*Suitable for people  Male  Female  All

\*effects  clean  Oil control  Pore shrinkage  Replenishment  Moisture  Lock water  skin whitening  Light spot  Sunscreen  Rejuvenation  Oxygen injection  compact  Anti wrinkle  antifading  Repair  Soothe  acne treatment  Detoxification  Eye care  repair  antioxidant  anti free radical  collagen  water oil balance  brighten skin color  fade acne print  pouch  eye lines  dark circles  lymphatic detoxification

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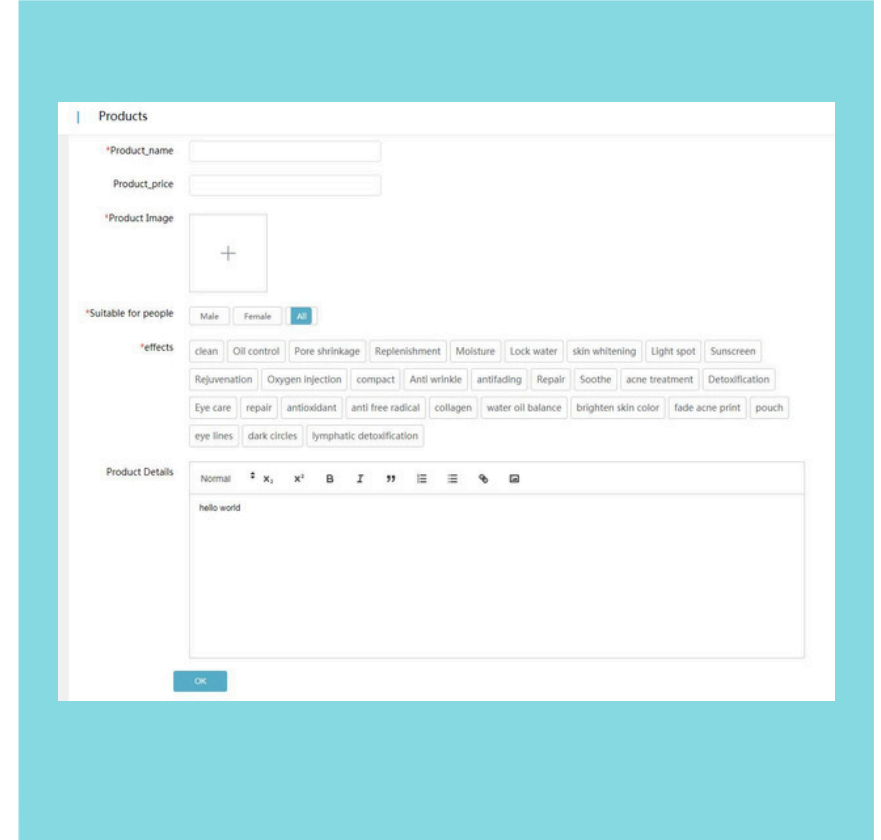
The screenshot shows a web-based configuration interface for a product. The interface is titled "Products" and contains several input fields and options:

- Product\_name**: A text input field.
- Product\_price**: A text input field.
- Product Image**: A placeholder for an image, indicated by a plus sign (+).
- \*Suitable for people**: Radio buttons for "Male", "Female", and "All".
- \*effects**: A grid of buttons representing various effects, including: clean, Oil control, Pore shrinkage, Replenishment, Moisture, Lock water, skin whitening, Light spot, Sunscreen, Rejuvenation, Oxygen injection, compact, Anti wrinkle, antifading, Repair, Soothe, acne treatment, Detoxification, Eye care, repair, antioxidant, anti free radical, collagen, water oil balance, brighten skin color, fade acne print, pouch, eye lines, dark circles, lymphatic detoxification.
- Product Details**: A rich text editor with a toolbar (Normal, Bold, Italic, Underline, Link, Unlink, Bulleted List, Numbered List, Indent, Outdent, Undo, Redo) and a text area containing "hello world".
- OK**: A button at the bottom right of the form.

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The image displays three overlapping screenshots of the EAST MEDICAL back-end management system configuration interface. The interface is organized into sections: Basic, Text, and Other. A vertical sidebar on the left contains navigation icons for Overview, Products, Reports, and Config. The top right corner shows the company name and language settings.

### QR code config

QR code configuration: QR code generated on H5 report page: [dropdown menu]

[OK] [Reset]

### surface detection

analysis dimension	analysis result	result Overview	result advice	Operation
<input type="checkbox"/> Blackhead	1 level care	Blackheads are the result of the oxidation of sebum inside the hair follicle opening by air and dust, mainly related to the high secretion of hormones. Daily life should pay attention to cleanliness and skin care, which can inhibit the formation of blackheads, reduce the sugar intake, and drink less sugary drinks such as milk tea and cola.	Pay attention to sun protection, moderate clean skin, keep the local skin clean and dry, you can use moisturizing skin care products, eat a light diet do not eat food with too high a salt content; eat more fresh fruits and vegetables, do not eat spicy and stimulating food. Reduce their own stress, appropriate to increase exercise, and develop good habits of work and rest.	[Edit]
<input type="checkbox"/> Blackhead	2 level care	The T-zone of the face is a high incidence of blackheads, as the sebaceous glands are active and secrete more oil that needs to be drained out through the skin pores. If not cleaned properly or too much sebum is secreted, a mixture of old keratin that is shed but not metabolized in time clogs the opening of the hair follicle. After being oxidized at one end in contact with the outside world, blackheads are formed.	Change your diet, take more beans, mushrooms, or dark green vegetables to replenish vitamin B and alkaline food, which can effectively inhibit oil production and reduce the growth of blackheads. Reduce the sugar intake, do a good job of moderate cleaning, avoid hand squeezing, you can use a cleaning mask once a week, with the use of acid products that can have unclogged pores, inhibit the production of blackheads.	[Edit]
<input type="checkbox"/> Blackhead	3 level care	Blackheads are the product of oxidation after the skin secretes excess sebum and old keratin mix. Usually, you should develop the habit of regular exfoliation and professional deep cleansing treatment every week, which can help remove skin dirt and grease, thus reducing the blackhead generation.	Daily care to remove makeup thoroughly, if makeup removal is not complete, not only will clog the pores, resulting in large pores, in addition to causing large pores, will also allow the growth of blackheads, pay attention to sun protection, often do care, avoid hand squeezing blackheads, can choose hydration and moisturizing, anti-oxidant effect of skin care products to slow the formation of blackheads.	[Edit]
<input type="checkbox"/> Blackhead	4 level care	Skin with visible blackheads are formed due to the keratinization and narrowing of the openings of sebaceous glands and the increased secretion of sebum contents, but they cannot be discharged effectively and smoothly from the follicular pores and accumulate inside the hair follicles, which are related to endocrine and dietary, as well as local cleanliness and hygiene factors of the skin.	Regular deep cleansing of the skin, and to avoid exposure to the sun, you can use a weekly cleansing mask, with the use of acid products that can gently adjust the keratin, unclogging the pores, do not squeeze by hand, easily lead to bacterial infection. Diet should not eat sweet, greasy, spicy and stimulating food, eat vitamin-rich fruits and vegetables to reduce skin oil secretion.	[Edit]

### content settings

chart display type:  column chart  Rose chart

skin color display:  5-level skin color classification  unevenness classification

skin age:  AI prediction algorithm  skin evaluation algorithm

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